

SEGUFIX®-Sheepskin Padding for SEGUFIX®-Standard



Item 8012

Consisting of:

1 SEGUFIX®-Sheepskin Padding for SEGUFIX®-Standard

Sizes: S / M / L

(corresponding to the size of the SEGUFIX®-Standard)



Only compatible with SEGUFIX®-Standard with Crotch Strap / Thigh Straps

SEGUFIX®-Pull-up Line



Item 8016

Consisting of:

1 SEGUFIX®-Pull-up Line




The SEGUFIX®-Pull-up Line (white label) is made from a cotton/rayon mix.

Table of Contents	Page
SEGUFIX®-Sheepskin Padding for Standard – Instructions	2 - 3
SEGUFIX®-Sheepskin Padding for Standard – Instructions for Care, measurements	3
SEGUFIX®-Pull-up Line – Instructions	4
SEGUFIX®-Pull-up Line – Instructions for Care, measurements	4
SEGUFIX®-Safety Information	5 - 8
Imprint	8

Item 8012

Intended use:

- More comfort for patients while positioned in bed
- Only compatible with SEGUFIX®-Standard with Crotch Strap / Thigh Straps


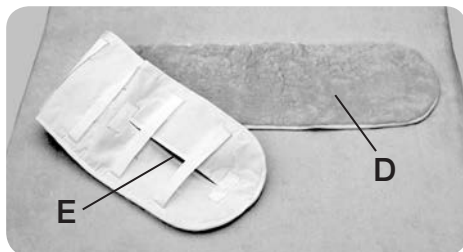


Instructions for use item 8012

For detailed instructions please refer to the respective instructions of the SEGUFIX®-Standard with Crotch Strap / Thigh Straps.

How to attach the SEGUFIX®-Sheepskin Padding to the waist belt of the Standard:

- Insert the pin of the lock (1) through one of the eyelets of the pin flaps on the waist belt of the SEGUFIX®-Standard (A). The metal pin must be in the center of the closed waist belt.
- With the fleece facing up, lay the Sheepskin Padding (D) onto the open waist belt (A), so that the slit (E) is on the same side as the loop (F) with the red stitching.
- Turn one side of the Sheepskin Padding over, and pull the waist belt through the three white loops (C) on the back of the Sheepskin Padding.
- Then also pull the side positioning strap (B) through the first white loop (C).
- Install the other side in the same manner.
- The waist belt is now ready to be used according to the instructions.

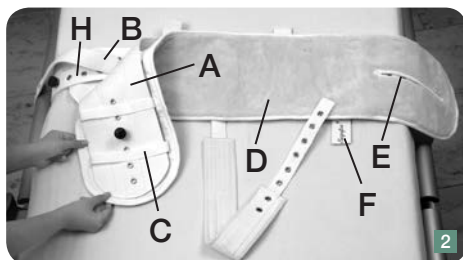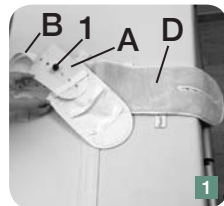


Follow SEGUFIX®-Safety Information and Warnings on pages 5 - 8

How to release the Sheepskin Padding from waist belt and bed:

For instructions on how to open the respective locking systems refer to the instructions "Locks and Keys".

- To release open the lock (1) of the waist belt (A) and lift all parts off of the pin.
- Then open both locks (2) of the attachment straps (H), and pull the side positioning straps (B) from the loops (G) of the Standard.

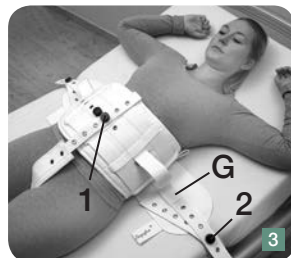

- Pull the waist belt (A) and the side positioning straps (B) out from the loops (C) of the Sheepskin Padding (D), and remove the Sheepskin Padding from the Standard.

WARNING: Read the Safety Information prior to using the SEGUFIX®-Sheepskin Padding.

- Cannot be used with SEGUFIX®-Standard with Crotch Strap / Thigh Straps size XL.
- The Sheepskin Padding is not compatible with products with Velcro® fastener.
- When using the SEGUFIX®-Sheepskin Padding with the SEGUFIX®-Standard with Crotch Strap (figure 3) or SEGUFIX®-Standard in combination with SEGUFIX®-Shoulder Restraint an additional lock is required.
- For more product info visit us online at www.segufix.com or www.segufix-shop.com

SEGUFIX®-Sheepskin Padding			Length of Padding		Width of Padding		Waist girth	
item	size	kg	mm	inches	mm	inches	mm	inches
8012	S	0,20	960	37.8	195	7.7	520 - 820	20.5 - 32.3
8012	M	0,28	1310	51.6	245	9.7	800 - 1200	31.5 - 47.2
8012	L	0,42	1620	63.8	245	9.7	1160 - 1560	45.7 - 61.4

Tables: approximate measurements – about 8 % shrinkage possible (products are not pre-washed)

Sheepskin - Instructions for Care:

Be sure not to leave sheepskin in water too long (do not soak).

Washing machine:

- Washable at warm wash cycles up to 30 - 40 °C, no spin-cycle, only use wool detergent

Hand washing:

- Wash quickly in lukewarm water (30 - 40 °C), only use wool detergent
- Rinse thoroughly with lukewarm water and gently spin out water.

How to dry the sheepskin:

- Hang dry or tumble dry on delicate cycle at low temperature. Important: Avoid direct sunlight!
- Before the sheepskin is completely dry, pull to reshape it and if needed, brush with a sturdy brush.

Positive qualities of natural sheepskin:

- Medical sheepskins are natural products which help to ensure a healthy and constant temperature. This gives a relaxing and revitalizing effect over the entire applied area. Body toxins secreted with perspiration are broken down.
- Immediate absorption of perspiration and seven times faster dispersion into air than synthetic materials.
- Sheepskins are a natural product and therefore have a self-cleaning effect.
- Natural sheepskin is at body temperature and never feels cold. Natural breathing of the patient's skin is ensured.
- Temperature balanced, suitable for summer and winter use.
- Non-static.

Doctors recommend natural sheepskin especially for use with bedsores, allergies, neuralgia, nervousness, poor circulation, rheumatism, sciatica, back pain and limb pain.

SEGUFIX®-Pull-up Line

Item 8016




Intended use:

- Helps the patient to sit up in bed in an ergonomic manner, preventing damage to the spine



Instructions for use item 8016

How to attach the SEGUFIX®-Pull-up Line to the bedframe:

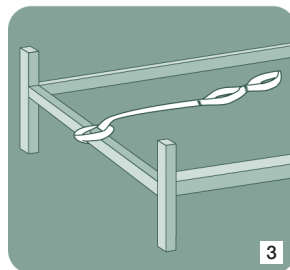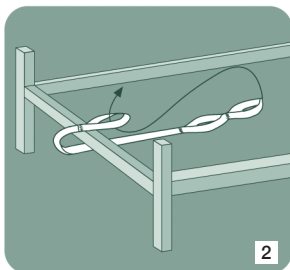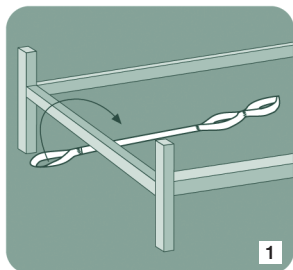

Follow SEGUFIX®-Safety Information and Warnings on pages 5 - 8

WARNING: Read the Safety Information prior to using the SEGUFIX®-Pull-up Line.

Item	Length		Width		Weight
	mm	inches	mm	inches	kg
8016	1600	63.0	48	1.9	0,18

Table: approximate measurements - about 8 % shrinkage possible (products are not pre-washed)

Care Instructions:

SEGUFIX®-Restraints (white) are made of a cotton/rayon mix and are washable in hot wash cycles up to 95 °C. Lower temperature and laundry nets will prolong product life.

Avoid methods that might bend metal parts of the system (e.g. laundry line, dry pressing, rotary irons), eyelets may be damaged.

Not pre-washed, up to 8 % shrinkage. Store in a dry place.

Detergents: non-aggressive, no fabric softener, no bleach.

Dryer: delicate cycle at low temperature and preferably in the laundry net.

Disinfection: may be disinfected using chemothermal disinfection washing procedure.

Overdosing can damage the product.



The following instructions for use are only applicable to original SEGUFIX® products. Do not use the SEGUFIX®-Restraint and Positioning System until you have read and understood the instructions for use.

Only use up-to-date instructions and keep them in a safe and readily available location. Product and instruction must match. Failure to use this product properly can cause serious injury or death. Your patient's safety depends on your using the SEGUFIX® system correctly.

1. Requirements for use

It is the user's responsibility to comply with all national laws. Always consider restraining protocols of your work place and current legislation. The instructions for use must be adhered to.

SEGUFIX® products may only be used by individuals with proper training and knowledge of the product. Legal provisions and doctor's orders must be strictly observed! The attending physician must ensure, that the patient's medical condition allows the application of the SEGUFIX® system.

2. Intended use

SEGUFIX® products may only be used for the medical purposes, which they were designed for.

3. Remove dangerous objects

Remove all objects from the reach of the patient, which could lead to injury or may damage the belts (e.g. glasses, jewelry and sharp objects).

4. Inspect products before use



Inspect before each use for broken stitches, torn, worn out or frayed parts and damaged eyelets. Damaged products could malfunction and cause injury. Do not use damaged or worn out products.

5. When opening, locks may not be under tension

When opening locks, ensure movement between the upper and lower points of contact (item 1206). Therefore each lock can hold up to a maximum of 4 belt layers (exception: products with "r" or "f" in the item number have thicker belts. These allow for a maximum of 3 belt layers!). When using the Patent Twist Lock (item 1306) use no more than 4 belt layers. When using the Push Lock (item 1406) use no more than 3 belt layers.

6. Always attach Side Positioning Straps

The SEGUFIX®-Standard must always be used with the Side Positioning Straps attached. Side Positioning Straps prevent the patient from helicoptering in bed or from getting over the edge of the bed causing compression of the chest, possibly resulting in death.

In some cases it is possible that patients get over the edge of the bed even with the Side Positioning Straps attached and the side rails in the up position. In these cases additional restraint measures must be taken.

Important: Insert the Side Positioning Straps through the respective loops (according to instructions) before securing them to the bed belt.


7. Beds, chairs and stretchers

- Only use SEGUFIX® products on beds, stable chairs and stretchers that allow the product to be attached to, in accordance with the instructions.
- Products must be attached in such manner, that they cannot get out of place.
- Note that straps and moving mechanics of beds, stretchers or chairs (or alternating pressure pads and the like), may interfere with each other causing damage, loss of function and injury.
- The attachment points must be free of sharp edges.
- SEGUFIX® products do not prevent beds, chairs or stretchers from tipping over or from unintended movement.

Helpful hints: European standards for beds: DIN EN 60601-2-52

European standards for wheelchairs: DIN EN 12182 and DIN EN 12183

8. All side rails must be in the up position

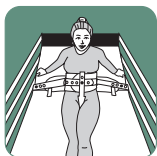

To prevent accidents always lock side rails in the up position (exception: 5-point-restraint).

Caution: Do not use with parted side rails except with a gap protector that effectively prevents the patient from slipping through the gap. An open gap could allow the patient to slip through even with the restraint fastened possibly causing compression of the chest.



9. Monitor patients in restraints

Patients in restraints require frequent monitoring. Aggressive, agitated or restless patients and patients in danger of aspiration require constant monitoring.



10. Test closure-systems prior to each use

Check SEGUFIX®-Magnetic Locks black/segufix.com, SEGUFIX®-Patent Twist Locks green, Push Locks, Snap Locks, Velcro® Fasteners and Buckle Locks for a secure hold. Do not submerge locks in any type of liquid. This may cause malfunction. If damaged locks fail to open, straps



can be cut with scissors or belt cutters. Check the head of the pin for signs of wear and tear (rounding off or deformation).

11. Pacemaker Warning



A distance of 10 cm (4 inches) must be maintained between a patient's pacemaker and magnetic locks and keys to prevent the pacemaker from possible malfunction (alternatively use Push Locks). For possible interference of strong magnets with other patient implants please refer to manufacturer's instructions of these products.

12. Test products for correct attachment

Frequently check products for correct attachment. The straps must be secured to the frame of the bed (or the folding part of the frame of the bed) in such a way, that they cannot slide out of place.

13. Only use original SEGUFIX® products

For safety reasons SEGUFIX® products must not be altered or be combined with third party products.

14. Use of SEGUFIX® products in vehicles



The SEGUFIX®-Transport Belt System does not replace the vehicle's legally required seat belt system. SEGUFIX®-Transport Belts may only be used as additional support or for restraint of a patient. Make sure that the patient can be rescued quickly in case of emergency (e.g. clearly visible and handy belt cutters).

15. Secure but not too tight

The restraints must be fastened tightly, but not interfere with breathing or blood circulation. Loosely fastened products may jeopardize the patient's safety. Frequently check products for correct position.

16. Storage

To prolong product life store belts and locking systems in dry and dark conditions at about 20 °C.

17. Flammability of SEGUFIX®-Bed Restraints



According to DIN EN 71-2 section 4.3 non-flammable.

According to DIN 75200 the burning rate is zero.

However, products may not come in contact with cigarettes or be exposed to open flame, as this could permanently damage the products, and this could lead to injury.



18. Correct sizing

Choosing the right size is of essential importance. Make sure that the size matches body and waist sizes (refer to sizing tables). Products that are too small or too large compromise the patient's comfort and safety.

19. Disposal

Carton and plastic wrapping can be disposed of in recyclable disposal, defective products in household garbage.

20. Declaration of conformity

For the declaration of conformity please visit:
https://www.segufix.com/conformity_en.php

21. Expert recommendation to prevent accidents

(Prof. Dr.-Ing. U. Boenick, TU-Berlin)

1. Always use Side Positioning Straps
2. Lock non-parted side rails in the up position
(exception: five-point-restraint)
3. The use of additional restraint elements may be necessary to eliminate bed egress.

22. Ensure prior to each use

- Test locks for a secure hold
- Visual inspection of belts, straps and eyelets
- Ensure sufficient medical indication
- Check legal requirements

Edition 15 · 2019-06

© by SEGUFIX®-Bandagen, 2019

The general business and delivery terms apply:

https://www.segufix.com/PDFs/AGBs/Terms_and_Conditions.pdf

Product specifications subject to change.
Deviations from pictures in colour and shape, errors and amendments excepted.

All rights reserved. Reprints or distribution using electronic systems/media, in its entirety or in parts, with prior explicit authorization in writing only.

All requests concerning this must be directed at:

SEGUFIX®-Bandagen

Das Humane System GmbH & Co. KG

Allerbeeksring 33 · 21266 Jesteburg
Germany

Phone: +49 (0) 41 83 500-0

Fax: +49 (0) 41 83 500-200

E-mail: export@segufix.de

www.segufix.com

www.segufix-shop.com